MicroMega One RECI / MicroMega One RECI Procedure Pack / MicroMega One RECI Assorted NiTi Root Canal Instruments

Protocol

MicroMega One RECI

MicroMega K-Files n008 **One G** **MicroMega One RECI 20.04**

MicroMega K-Files n010 **One G** **MicroMega One RECI 25.04**

One Flare **MicroMega K-Files n015** **One G** **MicroMega One RECI 25.06**

MicroMega K-Files n020 **MicroMega One RECI 35.04**

MicroMega K-Files n025 **MicroMega One RECI 45.04**

Ø	%	L	Stop color	Ring color	RPM	Recommended Torque (N.cm)
One Flare	25	9	17	-	250-400	3
One G	14	3	21-29	○	250-400	1,2

Ø	%	L	Stop color	Ring color	CCW - CW
One RECI 20.04	20	4	21 - 25 - 31	●	170° - 60°
One RECI 25.04	25	4	21 - 25 - 31	●	170° - 60°
One RECI 25.06	25	6	21 - 25 - 31	●	170° - 60°
One RECI 35.04	35	4	21 - 25 - 31	●	170° - 60°
One RECI 45.04	45	4	21 - 25 - 31	○	170° - 60°

Instructions for use **EN**

1. Indication
One Flare flaring during non-surgical endodontics treatment.
MicroMega One RECI - shaping of root canal during non-surgical endodontics treatment.
One G, MicroMega K-Files: glide path of root canals during non-surgical endodontic treatment.
Use by a dental professional only.

2. Contraindications
Apart from children under 2 years of age (ethylene oxide used in the sterilization process), there is no contraindication to the use of MicroMega One RECI for endodontically treating a tooth by orthograde route.

3. Complications
In cases of complex anatomy, peri-operative risks (instrumental breakage, ledge, stripping, zipping, false path, perforation, ...) could occur and induce a risk of infectious processes.

4. Warning and precaution
The decision to use an endodontic instrument must be related to the clinical case expertise in particular for a canal anatomy considered as too complex.
Patients identified as having a risk of infectious endodontitis.
Contents Nickel and Titanium and should not be used on patient with known allergic sensitivity to these metals.
Respect the good dental practice in particular the use of a dam and gloves.
Use in reciprocating motion at the recommended angles of rotation.
In case of complex anatomy, the maximum torque can be different from the value recommended by the manufacturer.
Use according to the procedure recommended (57).
Do not use in retreatment.
Instruments supplied sterile. Check the integrity of the packaging before use. If damaged, do not use the instrument. Respect the expiry date.
Use theUDI information which is only on the labelling until the last use.
Do not use if doubt on the respect of storage conditions.
Single use instrument. Do not reuse the instrument.
Otherwise the risk is a decrease of performances and security characteristics (fracture of the instrument in the tooth).
In case of doubt concerning the product identification, do not use it.
Check the instrument state before the use between each canal. If the instrument is damaged or shows signs of wear, do not use the instrument.
Inform the manufacturer and the country regulatory authority of any serious incident related to the instrument.

5. Clinical claims
MicroMega One RECI in normal conditions.
Clinical performance:
Shaping ability, adapted debris removal and cutting efficiency in curved canal anatomy.
Geometry adapted for irrigation.
Maintains the apical foramen in its original position.

6. Characteristics
MicroMega K-Files:
Diameters: 10 (purple handle), 15 (white handle), 20 (yellow handle), 25 (red handle).
Taper: 02 (red stop).
Material of operative part: Stainless steel.
Lengths: 25 mm.
Type 2 continuous rotatory instruments under EN ISO 3630-1:2019 standard.
Maximum recommended torque: see table "Recommended Torque".

7. Protocol
EWL: Estimated Working Length.
Operating procedure:
Progressive movement in three waves (3 up-and-down movements) with upward circumferential brushing movement. Insert the rotating instrument into the root canal until a resistance can be felt. Perform a circumferential brushing movement when feeling the resistance in order to eliminate the primary constraints. Remove the file from the root canal, clean the grooves, irrigate pass a K-Files 010 permeability test and reintroduce the rotating instrument to continue the progressive downward movement. On average, 2 to 3 passages are required before reaching the apex.

8. Storage and transport conditions
Material of operative part: Nitinol.
Use in connection with an endodontic contra-angle according to EN ISO 1797:2017 (Type 1).
Number of uses: maximum of 5 canals recommended on the same patient.
Medical device class according to Directive 93/42 and MDR 2017/745: IIa.
9. Disposal
After use, instruments must be placed in a secure container, used to collect cutting or slicing instruments (like needles or disposable bistouries) as per good dentistry practices.

10. Symbols
Nitinol raw material SST Stainless steel material
Quantity Root canal preparation
CLASSICS Hand Assortment
Continuous rotation Do not use if packaging is damaged

11. Symbols
NITI NiTi raw material SST Stainless steel material
Quantity Root canal preparation
CLASSICS Hand Assortment
Continuous rotation Do not use if packaging is damaged

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