
MicroMega One Glider

Reference: 30006062-A

	Ø	%	L	Stop	Ring	RPM	Maximum Recommended Torque (N.cm)
MicroMega One Glider	16	03	21 - 25 - 31	Yellow	White	250-400 rpm	1.2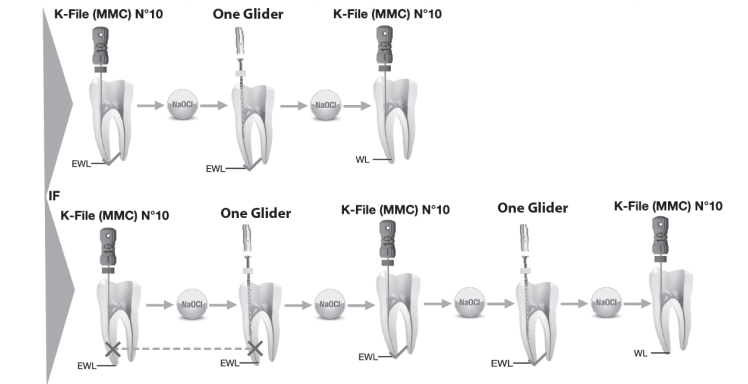

Instruction for use

1. **Indication**
Glide path of root canals during non-surgical endodontic treatment.
For use by a dental professional only.

2. **Contraindications**
Apart from children under 2 years of age (ethylene oxide used in the sterilization process), there are no contraindications to the use of MicroMega One Glider for endodontically treating a tooth by orthograde route.

3. **Complications**
In cases of complex canal anatomy, per-operative risks (instrumental breakage, ledge, stripping, zipping, false path, perforation, etc.) could occur and lead to a risk of infectious processes.

4. **Warnings/precautions**
- The decision to use an endodontics instrument must be based on the clinical case expertise, particularly where the canal anatomy is considered to be too complex.
 - Patients identified as having a risk of infectious endocarditis.
 - Contains Nickel and Titanium and should not be used on patients with known allergic sensitivity to these metals.
 - Respect the good dental practice in particular the use of a dental dam and gloves.
 - Use in continuous rotation at the recommended speed.
 - In cases of complex anatomy, the maximum torque may be different from the value recommended by the manufacturer.
 - Use according to the recommended procedure (§7).
 - Instruments supplied sterile: Check the integrity of the packaging before use. If damaged, do not use the instrument.
 - If applicable, respect the expiry date.
 - Keep the UDI information which is only on the labelling until the last use.
 - Year of manufacture: see labelling.
 - Do not use if doubt on the respect of storage conditions.
 - Single use instrument: Do not reuse the instrument. If you do, there is a risk of decreased performance and security characteristics (fracture of the instrument in the tooth).
 - In case of doubt concerning the product identification, do not use.
 - Check the condition of the instrument before use between each canal. If the instrument is damaged or shows signs of wear, do not use it.
 - Inform the manufacturer and the national regulatory authority of any serious incident relating to the instrument.

Packaging symbol	STERILEEO	⊗	Sterilized using Ethylene Oxide. Do not re-use
------------------	-----------	---	--

5. **Clinical claims**
MicroMega One Glider in normal conditions.

- Clinical performance:
- Shaping ability: adapted debris removal and cutting efficiency
 - Respect of canal anatomy
 - Geometry adapted to irrigation
 - Maintains the apical foramen in its original position

Usability:
• Secure progression and limit the interference

- Safety:
- Limited risk of breakage
 - Limited risk of shaping aberration
 - Limited risk of screwing

6. **Characteristics**
Type 4 rotating instruments under EN ISO 3630-1:2019 standard:
• Maximum recommended torque (not required): see table "Recommended Torque".

Material of operative part: Nitinol.

Use in connection with an endodontic contra-angle according to EN ISO 1797:2017 (Type 1).
Number of uses: single use. 4 canals maximum recommended on the same patient
Medical device class according to Directive 93/42/EEC and MDR 2017/745: IIa.

7. **Protocol**
General information on the protocol for use:

After opening the access cavity, sodium hypochlorite must be constantly used and renewed before and after each instrument. The sodium hypochlorite must be placed using a syringe in order to be injected as close as possible to the apical region.

Operating procedure :

MicroMega One Glider is introduced into the canal with a slow and regular movement without any apical pressure until the working length is reached.

8. Reprocessing instructions

Not applicable.

9. Storage and transport conditions

10. **Disposal**
After use, instruments must be placed in a secure container, used to collect cutting or sticking instruments (like needles or disposable bistouries) as per good dentistry practices.
11. **Symbols**

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