


# One Curve mini / One Curve Procedure Protocol Card

Visit our website for all the latest news:

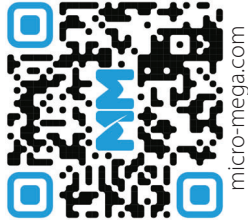

Follow us on socials!



@micromegasa




COLTENE  
Micro-Mega




@micromegasa

# One Curve mini / One Curve Procedure Protocol

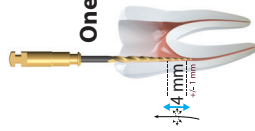


**One Flare**

250-400 rpm  
Max torque 3 N.cm

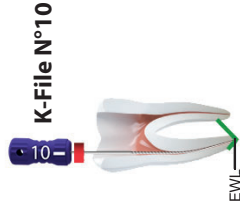


**K-File N°10**

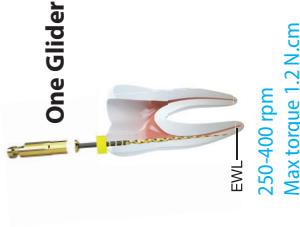


**One Glider**

250-400 rpm  
Max torque 1.2 N.cm

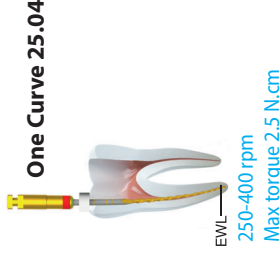


**One Curve 25.04**

250-400 rpm  
Max torque 2.5 N.cm

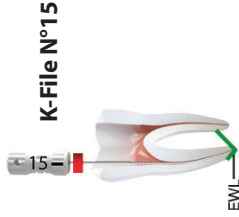


**K-File N°15**

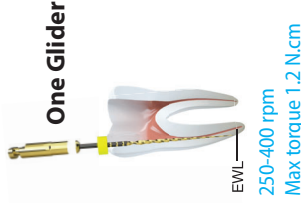

**One Glider**

250-400 rpm  
Max torque 1.2 N.cm

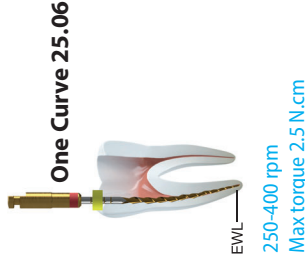

**One Curve 25.06**

250-400 rpm  
Max torque 2.5 N.cm

